Enjoy the moment  
with confidence  
The Essure® Confirmation Test

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Visite [essure.com](http://essure.com)  
1-877-ESSURE-1 (1-877-377-8731)

essure®  
YOUR FAMILY IS COMPLETE.  
YOUR CHOICE IS CLEAR.™

## Peace of mind you deserve

With Essure<sup>®</sup>, you can achieve the confidence you want in birth control. A simple test given three months after your procedure verifies your fallopian tubes are completely blocked. With the Essure Confirmation Test, you'll know you can rely on Essure as your sole form of birth control.

The test is performed by a doctor, usually a radiologist, and takes about 30 minutes. After that, you can start enjoying the moment—and stop worrying about unplanned pregnancy.

## Before the Essure Confirmation Test

The Essure Confirmation Test takes place three months after your Essure procedure to give your body time to form a natural barrier around the Essure micro-inserts. In rare cases, it can take longer than three months—up to six months at times—for the Essure micro-inserts to completely block the fallopian tubes. During this time, it is important that you continue to use another form of birth control, other than an IUD/IUC, to prevent pregnancy.

To prepare for your Essure Confirmation Test, talk to your doctor about whether you should take antibiotics or any pain medications before the test. You should also let your doctor know if you are allergic to shellfish, iodine, or contrast fluid.

## During the Essure Confirmation Test

While the Essure Confirmation Test is a type of HSG (hysterosalpingogram), it is different from the HSG used to diagnose and treat infertility—and is considerably more comfortable. In fact, most patients return to normal activities right after the test.

Here's what to expect:

- Just like your annual exam, you will be asked to lie on a table. A speculum will be inserted into your vagina to gently expand the opening.
- Then a thin tube will be passed through the cervix into the uterus. A small amount of dye will flow through the tube to the opening of the fallopian tubes.
- The radiologist will position the x-ray over your abdomen to verify that fluid cannot flow past the micro-inserts—ensuring your fallopian tubes are completely blocked.

## After the Essure Confirmation Test

For a few hours, you may feel some cramping similar to a normal menstrual period, which can usually be relieved with over-the-counter medication. You may also have a thick discharge as the dye is released from your uterus. Use pads, not tampons, until the flow has stopped.

After your Essure Confirmation Test, your doctor can provide further guidance and answer any questions you may have.

## Results

Your doctor will discuss the results of your Essure Confirmation Test with you as soon as they are available—sometimes even on the day of your test. Once your doctor informs you that your fallopian tubes are blocked, you can discontinue using your other form of birth control and rely on Essure with confidence.

# Considerations

There are few risks associated with the Essure Confirmation Test, and adverse reactions are rare. You should be aware that, because you will be receiving an x-ray, you will be exposed to very low levels of radiation, which is standard with most x-rays. Some rare but potential side effects of the Essure Confirmation Test include infection, spotting, and allergic reaction to the dye.

Please call your doctor if you experience:

- Severe or increasing pelvic pain
- Heavy vaginal bleeding (requiring more than a pad for one or two hours)
- Unusual vaginal discharge or odor
- A fever over 101°F (38.3°C)

You can find a complete list of risks and considerations on the Essure website at [essure.com](http://essure.com) or call the Essure Information Center at 1-877-ESSURE-1 (1-877-377-8731) to speak to a counselor about the Essure Confirmation Test.

Then rest assured with the knowledge and confidence that you'll never have to worry about unplanned pregnancy again.

## Essure Confirmation Test Appointment:

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essure® YOUR FAMILY IS COMPLETE. YOUR CHOICE IS CLEAR.™

essure® YOUR FAMILY IS COMPLETE. YOUR CHOICE IS CLEAR.™

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©2008 Todos los derechos reservados. Conceptus y Essure son marcas registradas y Your Family is Complete Your Choice is Clear es una marca de servicio de Conceptus, Inc. CC:117-02 07NOV08